



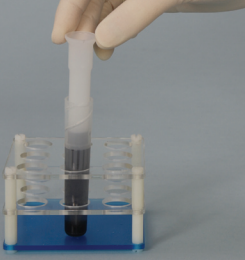


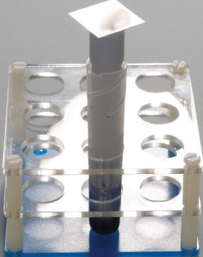



StatSpin® OvaTube®

Supplied By Vetlab Supplies Ltd
www.vetlabsupplies.co.uk

Instructions For Use

[Click Here For More Information About OvaTube](#)

<p>1</p> 	<p>2</p> 	<p>3</p> 
<p>OvaTube® – Tube, Mixer, and Filter</p>	<p>Add floatation fluid to the fill line.</p>	<p>Push Mixer into faecal sample and transfer to the Tube.</p>
<p>4</p> 	<p>5</p> 	<p>6</p> 
<p>Turn Mixer back and forth and gently squeeze bulb to mix sample. Discard Mixer.</p>	<p>Insert Filter into Tube aligning tab on filter with notch on tube. Slightly twist Filter clockwise.</p>	<p>Add floatation fluid to fill line Click Here For Details.</p>
<p>7</p>  <p>Insert Tube into horizontal or fixed angle centrifuge. Spin at recommended setting.</p> <p>If You Do Not Have A Centrifuge That Takes Large Tubes, Stand For 15 Mins In A Tube Rack</p>	<p>8</p>  <p>Transfer Tube to rack. Add coverslip. Turn Filter clockwise until fluid touches coverslip.</p>	<p>9</p>  <p>After 5 - 15 minutes, transfer coverslip to slide and examine under microscope.</p>