

StatSpin® OvaTube®

Supplied By Vetlab Supplies Ltd
www.vetlabsupplies.co.uk

Instructions For Use

[Click Here For More Information About OvaTube](#)



OvaTube® – Tube, Mixer, and Filter




Add floatation fluid to the fill line.



Push Mixer into faecal sample and transfer to the Tube.



Turn Mixer back and forth and gently squeeze bulb to mix sample. Discard Mixer.

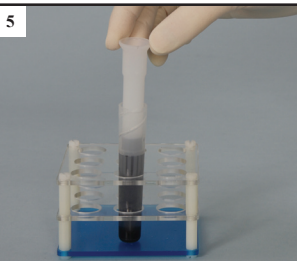

Insert Filter into Tube aligning tab on filter with notch on tube. Slightly twist Filter clockwise.




Add floatation fluid to fill line
[Click Here For Details.](#)



Insert Tube into horizontal or fixed angle centrifuge. Spin at recommended setting.

If You Do Not Have A Centrifuge That Takes Large Tubes, Stand For 15 Mins In A Tube Rack

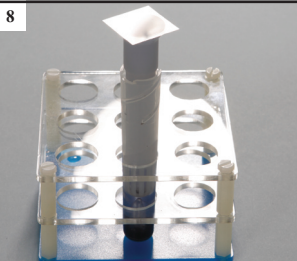

Transfer Tube to rack. Add coverslip. Turn Filter clockwise until fluid touches coverslip.



After 3 minutes, transfer coverslip to slide and examine under microscope.