


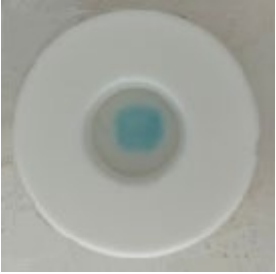


# Target equine test results interpretation.

Result Photo	Color	Progesterone ng/ml	Recommendation
	Bright Blue C1	0-1	No progesterone is present and there is no functional CL.
	Light Blue C2	1 - 2	Progesterone is either rising after heat or falling (approaching heat). See Figure below
	Faint Blue C3	2.1 - 5	If a previous test was bright blue, then a light blue result indicates progesterone is rising. The mare has ovulated 1 to 2 days before. If a previous test was white, then progesterone is dropping. The mare is approaching her estrus period and should be tested again in 2 days.

Result Photo	Color	Progesterone ng/ml	Recommendation
	White C4	> 5	<p>White result indicates a high progesterone level and a functional CL is present.</p> <p>Mare has ovulated and is in diestrus or in a non-fertile period.</p>